**Available in the following presentations**  
 Disponível nas apresentações **Disponible dans les présentations** Verfügbar in den Ausführungen **Deze klemmen zijn verkrijgbaar** Disponibile nelle versioni **Disponible en las presentaciones**  
 Finns i följande kategorier

**EN** STERILE

**PT** ESTERILIZADOS

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**IT** STERILE

**ES** ESTERILIZADOS

**SE** STERILA

**REF** 489-051

**1** x 110 x 18 = 1980 pcs

**CE** 0120 **STERILE EO**

**EN** **NON STERILE**  
 Sterilizable by steam and gas (Ethylene Oxide)

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## CAUTION

Incorrect placement of the clamp on the umbilical cord may lead to an inadequate seal and consequent blood loss that might prove fatal for some babies. Read the instructions before using the device.

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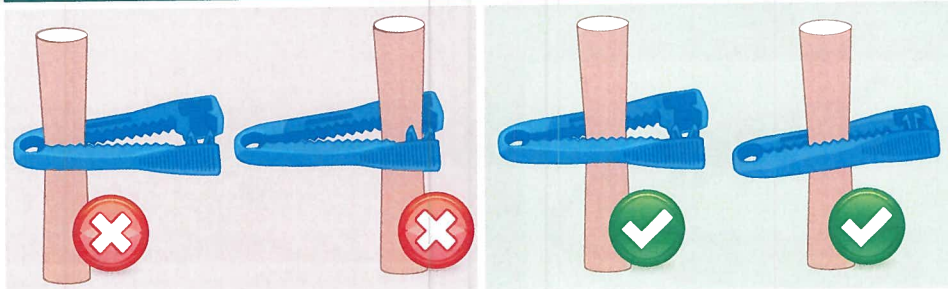
Fig. 1

**EN** For use only by trained medical staff. Discard after use according to the laws in force.

**⚠ Caution:** Incorrect placement of the clamp on the umbilical cord may lead to an inadequate seal and consequent blood loss that might prove fatal for some babies. Read the instructions before using the device.

- Always check the integrity of the locking system before use. Do not use if incomplete or blocked.
- Check the thickness of the umbilical cord. When dealing with thin cords it may be necessary to fold it before placing the clamp to obtain a bigger resistance to the clamp closing.
- Centre the cord within the clamp body (figure 1).
- Check that there is not any object blocking the locking site (eg. swabs, surgical instruments, etc.).
- Close the clamp until it locks and clicks (figure 2). Make sure that the clamp is well closed and that it is impossible to open it with the hands. Cut the umbilical cord.
- Confirm that there is no bleeding at the end of the cord.
- Check regularly for signs of bleeding due to a leaking clamp or a damaged cord.

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