SampLok Adapter Cap

INDICATIONS FOR USE (EN)

SampLok Adapter Cap (SAC) is intended for use to facilitate collection of blood sample into culture bottles or vacuum tubes. It is a single use device. There are no known contraindications for the SAC, though care should be taken per the warnings and precautions included in this FIU.

WARNINGS AND PRECAUTIONS

SampLok Adapter Cap is a single use device. Do not attempt to remove the needle after use.

INSTRUCTIONS FOR USE

- 1 Attach a collection needle to the SampLok Adapter Cap. For SampLok Adapter Cap without a pre-attached Luer, screw the blood collection needle into the opening at the top of the cap until it is fully seated. For SampLok Adapter Cap with a pre-attached Luer, insert the Luer tip into the hub of the blood collection needle until it is secure.
- 2 Skin preparation: Prepare and disinfect venipuncture site following your facility procedure.
- 3 Culture bottle preparation:

- 4 Remove the culture bottle protective cap (B). Disinfect culture bottle septum as per culture bottle manufacturer instructions (C).
- 5 Perform venipuncture following your facility procedure.
- 6 Open the SampLok Adapter Cap Safety Lid by pulling finger tips on the two tabs, pressing the upper tab down and the lower tab up (D). Fully open the Safety Lid so that the cap opening is clear (E).
- 7 Collect blood sample into culture bottle and/or vacuum tube as required. If using Cap Insert, place into SampLok Adapter Cap, align the tabs with the notches in the rim of the SAC and press the insert into the cap until there is a click indicating that the insert is in place (F).
a. Insert vacuum tube into the center of the insert (H).
b. Remove vacuum tube when filled.
c. Repeat steps 7a & 7b for each additional vacuum tube.
d. After filling the final culture bottle or vacuum tube, close the lid with your fingers or against a surface until the Safety Lid snaps closed.
e. Discontinue the venipuncture following your facility procedure.
- 8 Dispose of collection needle and SampLok Adapter Cap and Insert (if used) in a biohazard sharps container following your facility procedure.

Product Information






Carton quantity Carton weight

Medical Device

CAUTION: Federal (USA) law limits this device to sale by or on the order of a practitioner.

Corporate Headquarters
ITL Health Group
Tel: +61 (3) 9088 7391
info@itlhealthgroup.com

Sales Office
ITL BioMedical
Tel: +1 (888) 411 2851
sales@itlbiomedical.com
SampLok® is a registered trademark of ITL Corporation, Canberra, Australia.

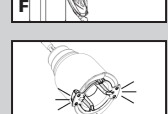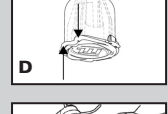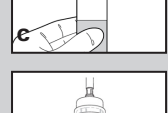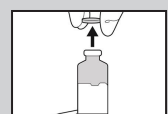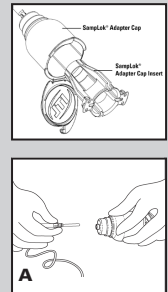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