English

Instruction for use

Sterile Lubricant Jelly

Description:
The Lubricant Jelly is sterile, clear, colorless, non-sticky, non-greasy, non-staining, non-irritating lubricant. It is water soluble, high viscosity gel-like liquid for use as a lubricating media for insertion of medical instruments during medical procedures. It is easily rinsed off with water.

Composition:
Purified water, Polyethylene Glycol (PEG), Carbomer, Methylparaben, Propylparaben, Sodium Hydroxide.

Variants:
Tubes: 4oz, 42g, 82g.
Packets: 2.7g, 3g, 5g, 9g, 15g.
Syringes: 8g.

Intended purpose:
The Lubricant Jelly is designed for medical purposes to lubricate body orifices, assisting in the insertion of devices for diagnostic and therapeutic purposes.

- Typical uses of the lubricating jelly include (but not limited):
 - in gynaecological ultrasound, and hospital procedures.
 - in insertion of rectal thermometers, enemas, douches, similar types of nozzles and diagnostic or therapeutic devices.
 - for Foley Catheter insertion procedures.
 - for lubrication prior insertion of catheters, endoscopes, and surgical devices.
- Ensure that Lubricant Jelly is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.

Intended users:
The intended users of the Lubricant Jelly are patients, their Carers and/or Healthcare Professionals.

Warnings and precautions:

- Do not use packet if its torn or cut.
- Do not use tube if the seal is broken or tube damaged.
- Do not use syringe if its damaged.
- Keep out of eyes and ears.
- Keep out of reach of children.
- Re-use of this medical device may result in infection and/or cross-contamination.
- Resterilization, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.
- Contents sterile in unopened, undamaged packaging.

Contradictions and side effects:
There are no known side effects associated with using Lubricant Jelly, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using Lubricant Jelly and consult your Healthcare Professional. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority (of the Member state in which the user is established).

Directions for use:
Packet: Sterile and single use. Tear packet at one end and squeeze to obtain desired amount. Apply as needed.

Tubes: Sterile until opened. Open the cap and remove the seal. Apply as necessary. Close cap after use.

Syringes: Sterile and single use.

Sterile devices:
Lubricant jelly products are supplied sterile. Sterilized by gamma after packaging process.

Storage and handling:

- Keep dry and out of direct sunlight.
- Do not use after expire date.
- Store at room temperature.
- Avoid excessive heat and humidity.

Disposal:
Lubricant jelly products must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

Contact and manufactured information:

- Sion Biotext Medical Ltd.** 3 Eshel St., Sapirim Industrial Park, 8701002, Sderot, ISRAEL. Tel: 972-8-6898650 Fax: 972-8-6898572 Email: sales@sn-medical.com
- Theo Manufacturing B.V.** Sleurweg 44, 6222 NK Maastricht, The Netherlands. Tel: +31 (0) 43 325 1773 Email: info@theomanufacturing.com

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