
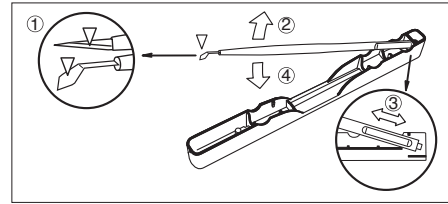
MICROSURGERY KNIFE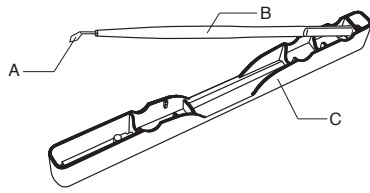

EN Instructions for use

1. Device Description

The device consists of blade (A) and handle (B), placed in a case (C).

The intended purpose of this sterile disposable ophthalmic microknife is to incise or penetrate eye tissue for the treatment of eye diseases. Lift the handle (2) to remove. To store, insert the back of the handle into the case with the blade tip facing down (1 and 3) and put the handle back in the case (4).

2. Instructions for Use

For use by healthcare professionals only.

Incise or penetrate the eye tissue.

Mind the position of the blade tip when removing it from the case and storing it.

After use, dispose of the device properly as medical waste to prevent infection.

3. Remarks

Do not use on patients who present sensitisation or allergy.

Reuse of this device presents a risk of infection to the patient.

Handle with care so as not to injure yourself by touching the edge of the blade.

If the cutting edge comes in contact with a foreign object, replace it with a new one.

Do not use the device if any damage or other abnormalities are observed.

Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority.

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KAI INDUSTRIES CO., LTD.
KAI EUROPE GmbH