**3M™ Tegaderm™ Transparent Film Dressing Frame Style**  
**3M™ Tegaderm™ HP Transparent Film Dressing Frame Style**  
**3M™ Tegaderm™ Transparent Film Dressing with Border**  
**3M™ Tegaderm™ HV Transparent Film Dressing with Border**  
**3M™ Tegaderm™ HP LV Securement Dressing**

**Product Description**  
The dressing consists of a thin backing with an adhesive with low allergenic potential not made with natural rubber latex. Dressings with borders are notched and reinforced with soft cloth tape to provide a better seal around catheters and other devices. The dressing is breathable, allowing good oxygen and moisture vapor exchange. It is waterproof and impervious to liquids, bacteria, and viruses. An intact dressing protects the skin from outside contaminants.  
\*In vitro testing shows that the transparent film provides a viral barrier from viruses 27 mm in diameter or larger while the dressing remains intact without leakage.

**Indications for use**  
The dressing can be used to cover and protect catheter sites and wounds, to maintain a moist environment for wound healing or to facilitate aseptic debridement, as a secondary dressing, as a protective cover over at-risk skin, to secure devices to the skin, to cover first- and second-degree burns, and as a protective eye covering. Do not use the dressing as a replacement for sutures and other primary wound closure methods.

**Warnings**  
Using the dressing to cover a drug delivery patch could lead to faster absorption, increasing the risk for overdose.  
Do not reuse. Remove only by infection or other illness injury.

**Precautions**  
1. Stop any bleeding at the site before applying the dressing.  
2. Do not stretch the dressing during application to ensure it does not cause skin trauma.  
3. Make sure the skin is clean, dry, and free of residue and lotion and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.  
4. The dressing may be used on an infected site, only when under the care of a healthcare professional.

**5. Antimicrobial dressings containing polyethylene glycols may compromise the strength of the 3M™ Tegaderm™ HP Transparent Film Dressing.**  
**Intended Use/Directions for Use**  
Intended users are healthcare professionals following the instructions for use and application instructions for the dressing as part of the dressing. For optimal adhesion, apply gentle firm pressure over entire dressing.

**Shelf Life/Disposal**  
For shelf life, refer to the expiration date, printed on each package.  
If the sterile packaging is damaged or unintentionally opened before use, discard the product, do not use.  
Note: Dispose product in facility protocol.

**Information for Users**  
Please report a serious incident occurring in relation to the device to the local and national competent authority (EU) or local regulatory authority.  
For further information, contact your representative or contact us at 3M.com and select your country.

**Parasetment filmtransparent av code 3M™ Tegaderm™**  
**Parasetment filmtransparent av code 3M™ Tegaderm™ HP**  
**Parasetment filmtransparent av bordure 3M™ Tegaderm™**  
**Parasetment filmtransparent av bordure 3M™ Tegaderm™ HV**  
**Parasetment de maintient 3M™ Tegaderm™ HP LV**

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